

Thank you for choosing Health By Genetics. The following instructions explain the simple process for taking a sample of your DNA from cells on the inside of your cheek. It is an easy and simple process that will provide a wealth of information to you and your providers. Please follow the instructions and contact us if you have any problems.

\*Do not eat or drink 10 minutes before collection.

- \*Avoid smoking or chewing tobacco for at least 30 minutes. Rinse mouth with water 10 minutes before collection.
- \*Avoid touching the collection-end of the swab with your fingertips.

## **Collection Steps:**

- 1. Uncap the top of the small vial, be careful not to spill the contents. Stand the vial up on a table or countertop.
- 2. Peel the stick end of the Swab Package open and remove the swab. **Do Not touch the cotton tip with your fingers**.
- 3. Vigorously rub and roll the swab on the inside of **EACH** cheek for 30 seconds. The objective is to gather cells from your cheek. You should use enough pressure during the swabbing action that it pushes your cheek outward.
- 4. When done doing this to both cheeks, place the cotton swab tip into the vial. There is a **Black Line** indicating a breakpoint, just above the cotton tip. With the swab tip down in the vial, bend the swab stick and break it off, leaving the cotton tip in the vial.
- 5. With the Swab tip in the vial, <u>tightly</u> recap the vial.
- 6. Fill out the include label with name and date of birth (DOB) and attach to the vial.
- 7. Review or complete the requisition form.
  - a. Include medication list, vitamins, supplements, and over-the counter medications.
  - b. Please ensure that the form is signed by a direct care provider and the patient or patient's guardian.
- 8. Place the sealed tube and the completed requisition form in the postage paid envelope.
- 9. Remove the silver covering strip and seal the envelope. Drop the envelope in any FedEx drop box or call FedEx to schedule a pick-up (FedEx)
- 10. In 7 to 14 days, your personal pharmacist will contact you to set up a 1:1 or virtual meeting to review your results.

Please call, text, or email if you have any questions about conducting the cheek swab.